

VIABLE MATRIX PLUS

BONE ALLOGRAFT

PREPARATION GUIDE



STEP 1

Open the pouch and remove the syringe. Confirm the syringe cap is securely locked.




STEP 2

Place the syringe in the ambient sterile bath for 10-15 minutes, ensuring the contents are submerged.




STEP 3

Remove the syringe from the sterile bath and unscrew the syringe cap from the top as shown above.

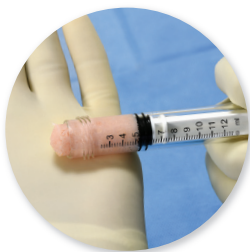

STEP 4

Apply slight pressure to the plunger and extrude product out of the syringe into the palm of your hand.

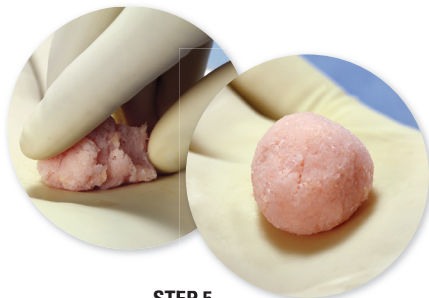

STEP 5

Viable Matrix Plus is now ready to mold to desired shape. Product remains viable for up to 4 hours on the back table in an operative field.